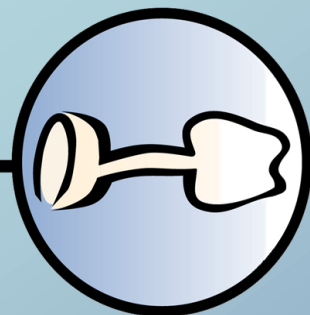
©TriplePress

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TriplePress

German HighTech Systems

...just better in dental solutions!

GPS Ceram
German HighTech Systems

CAMELION

...by TriplePress

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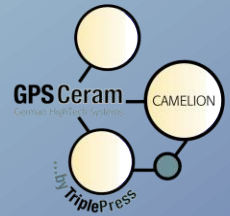
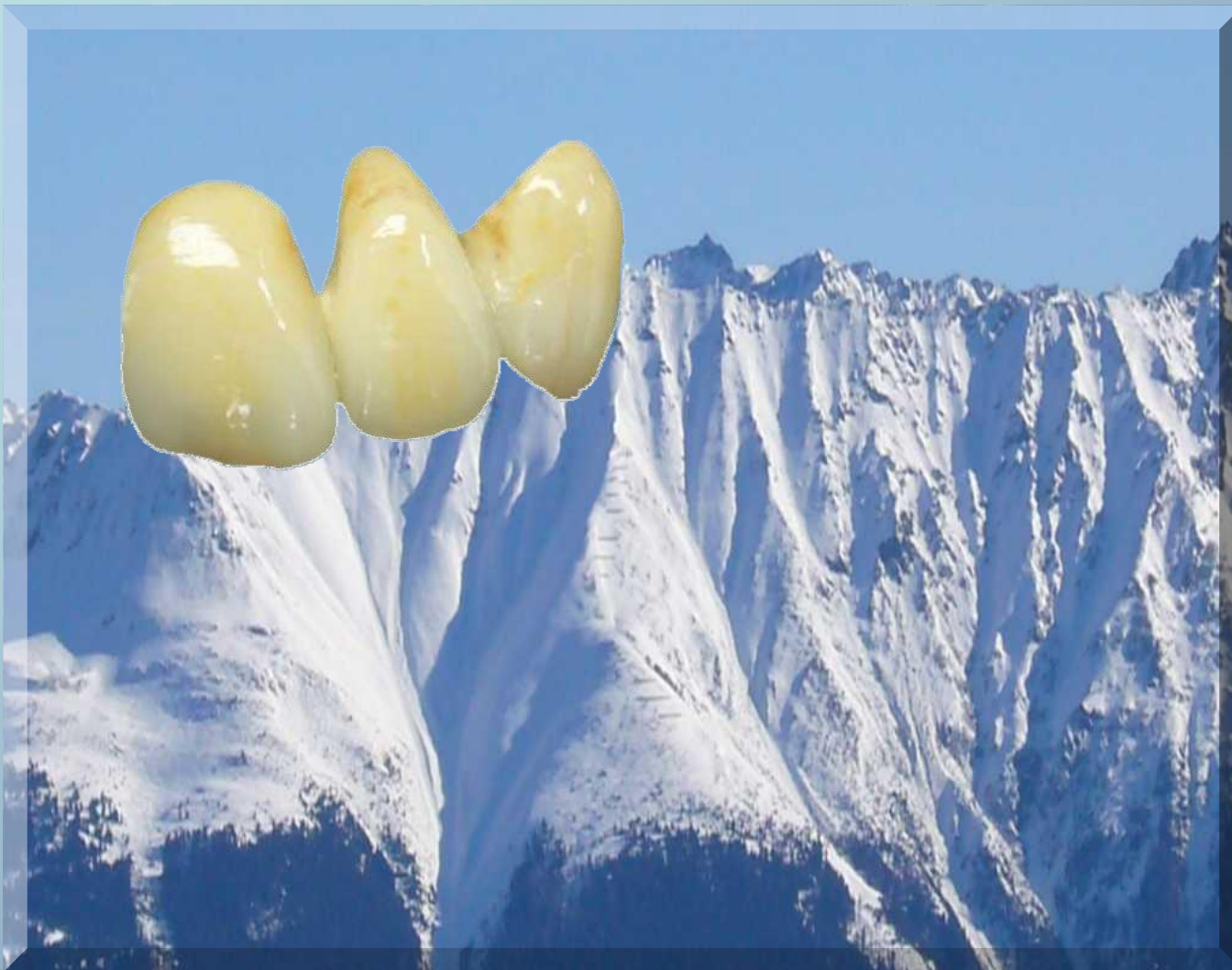
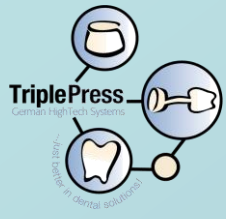
Instructions for technicians

supported by:
firingcharts
material-lists
& folder

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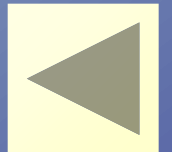
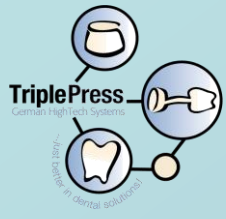
TriplePress
German HighTech Systems

TriplePress
...in dental solutions

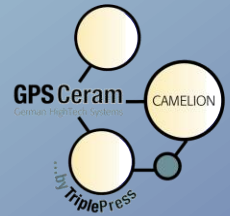
GO

SOUND

STOP

GPS Ceram
German HighTech Systems

TriplePress

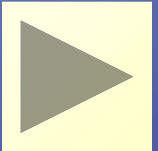
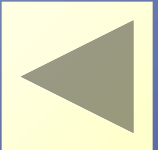
CAMELION

GPS Zirkon
German HighTech Systems



TriplePress

CAMELION



Quitt

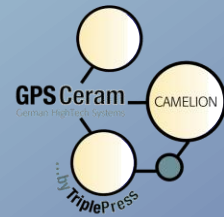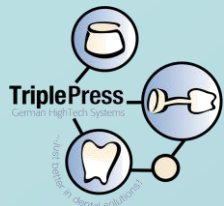Testkit

SOUND

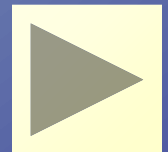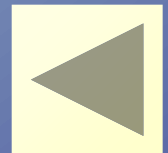
Gialloy CB

- Perfect to use with Triple-Press
- NP-metal for all cases
- Bonding-strength more than 50 MPa
- Nickel free


Triple-Vest

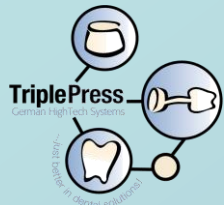- Excellent speed investment; over night investment
- For all alloys
- Investment for all cases
- Inlay, onlay crown and bridges
- For all pressable porcelains

TriplePress
German HighTech Systems

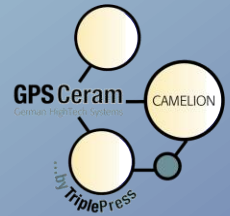...by TriplePress
...in dental solutions

GPS Ceram
German HighTech Systems

...by TriplePress

GPS Zirkon
German HighTech Systems

...by TriplePress

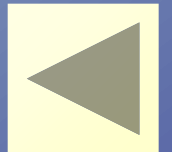
GO

SOUND




STOP




Quitt

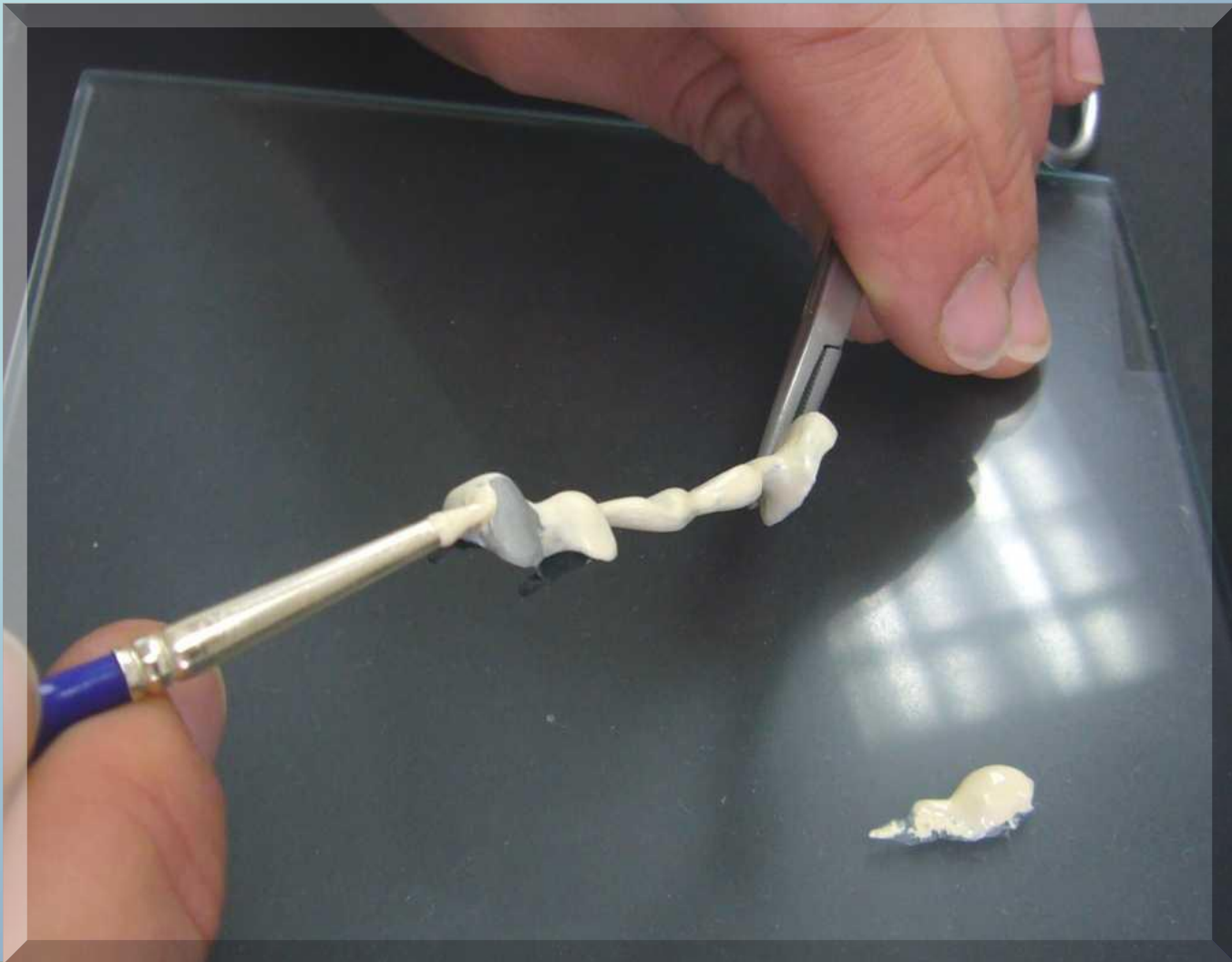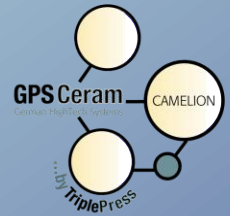
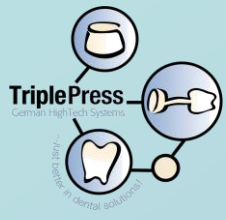
Camelion-Opaquer für alle Farben,
one and only for all cases

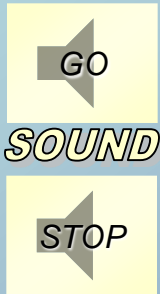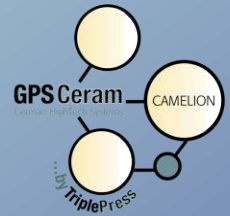
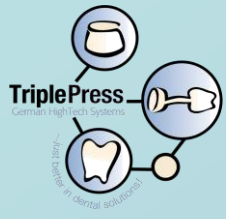
1. Opaquer 970°C
Opaquer applied onto the bridge

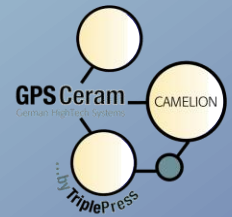
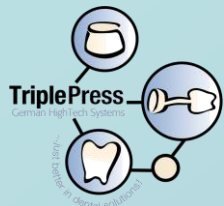
2. Opaquer 970°C
Opaquer applied onto the bridge

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




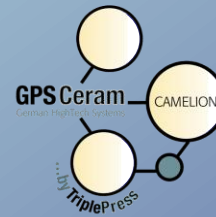
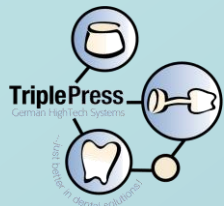
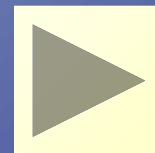
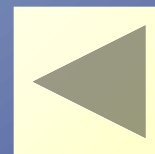
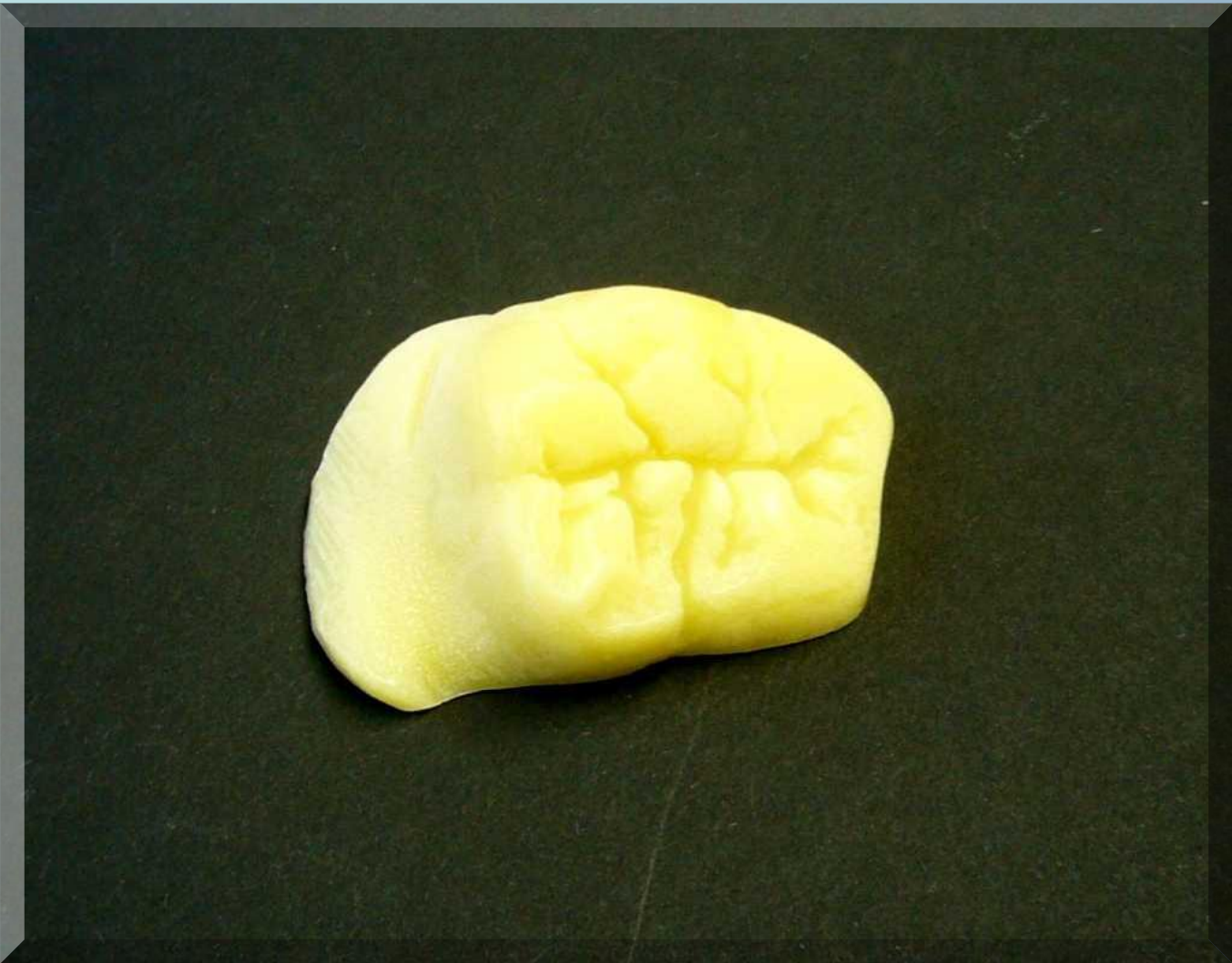
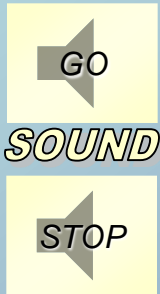
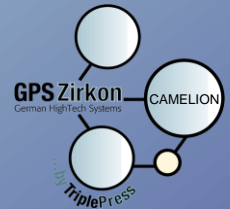
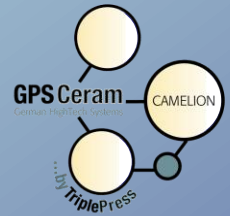
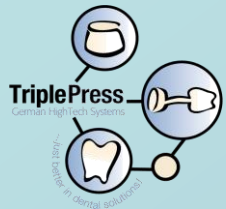
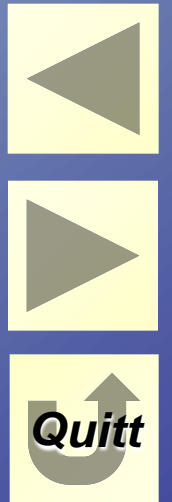
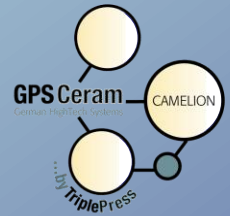
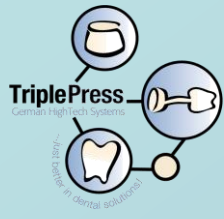
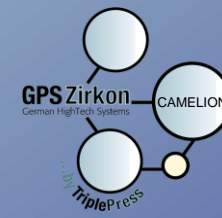
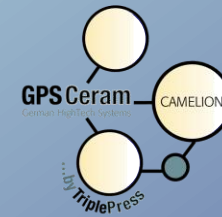
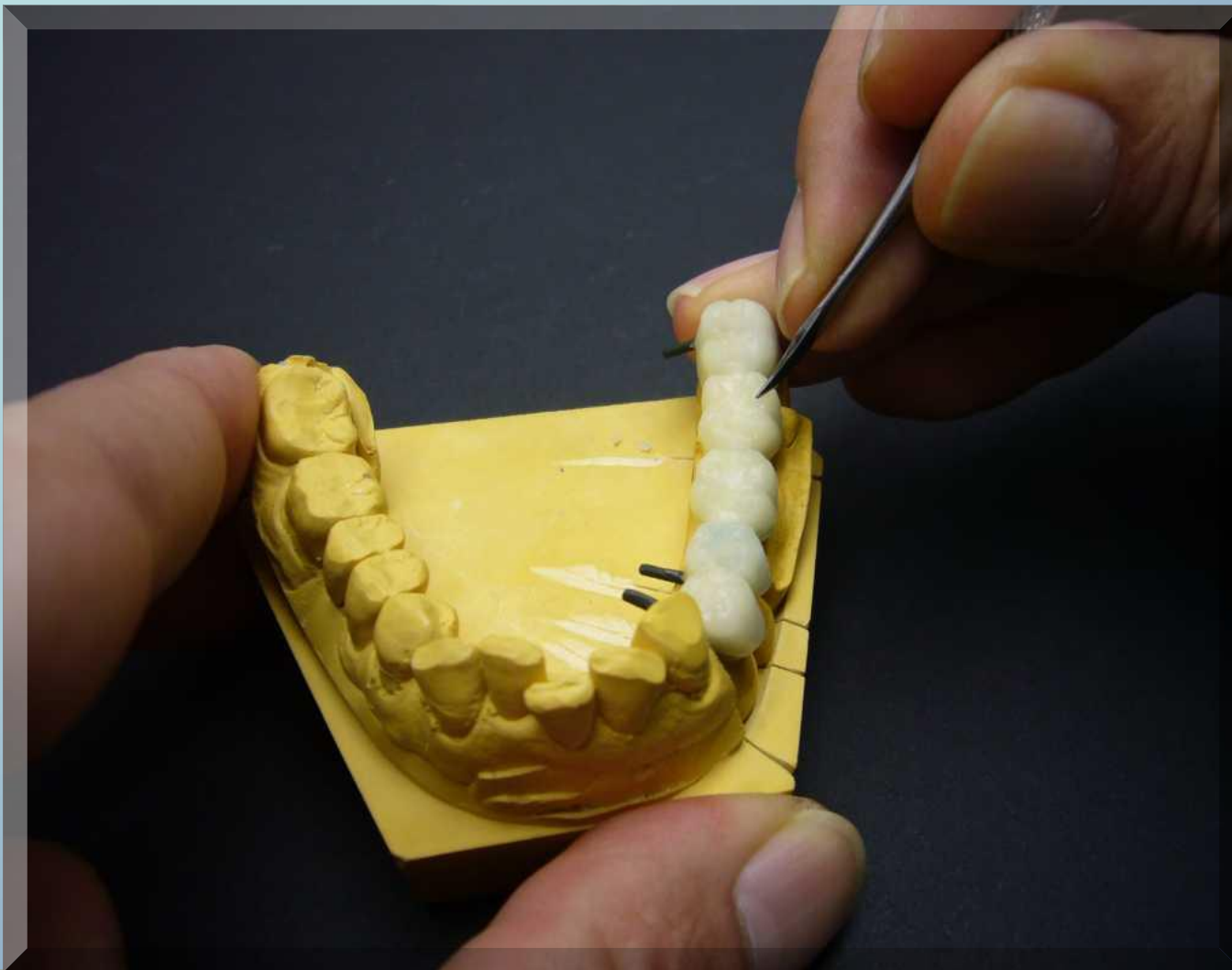
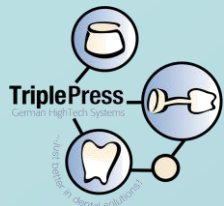
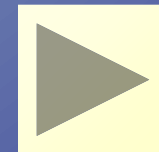
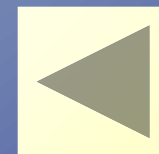
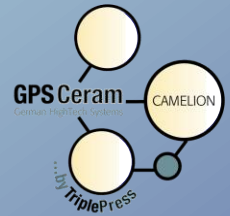
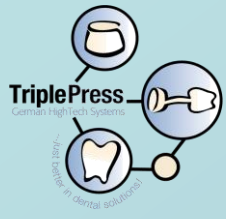
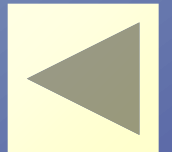
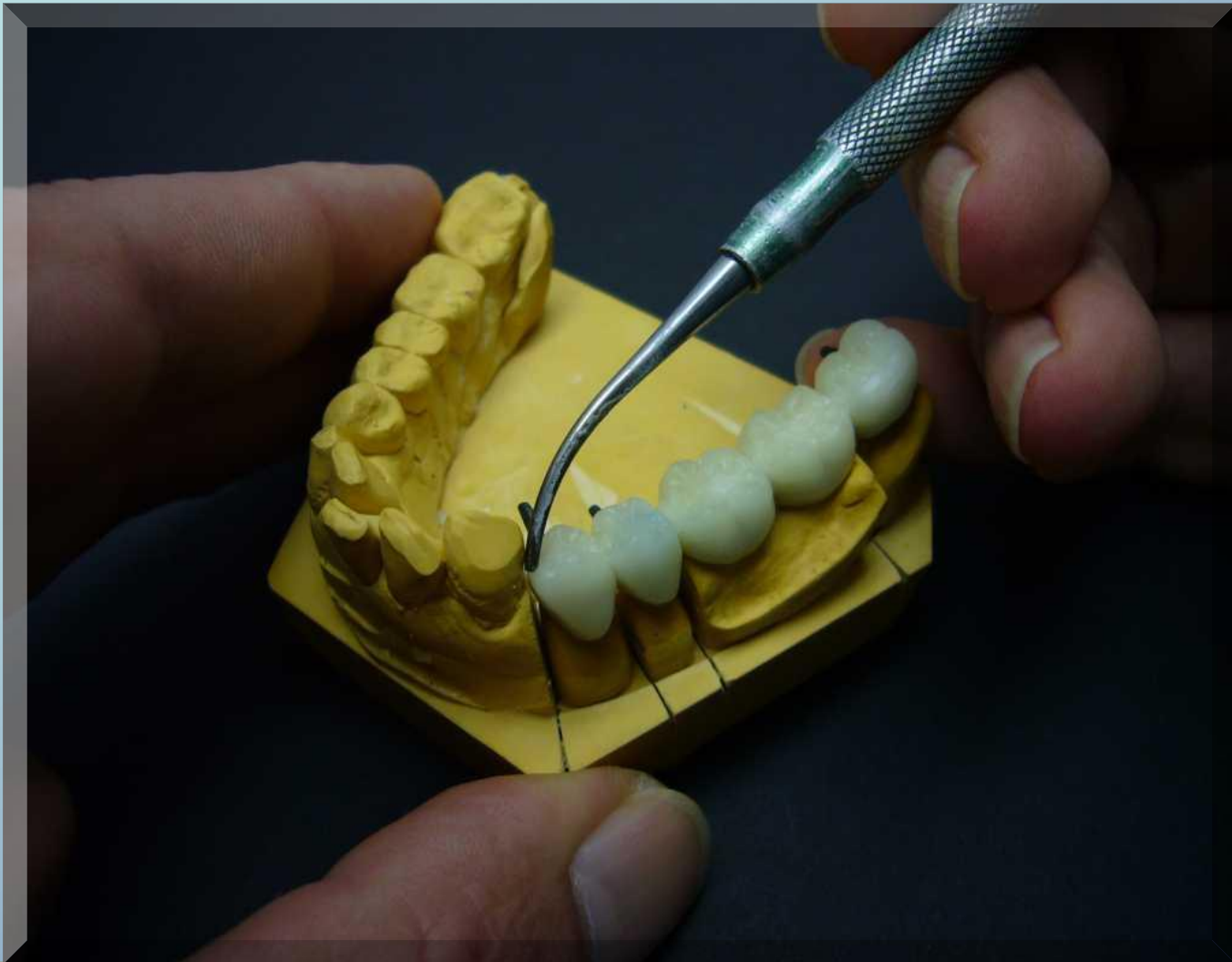
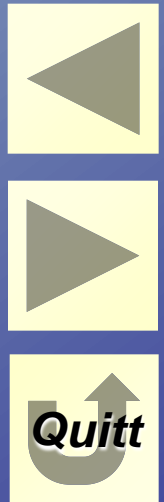
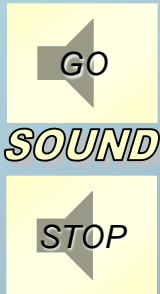
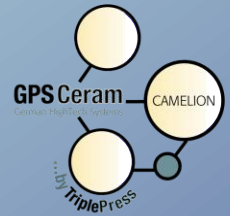
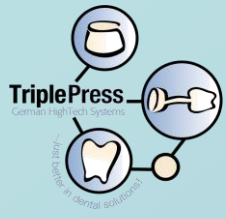
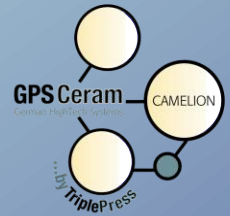
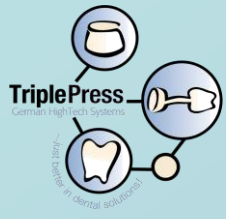
SOUND

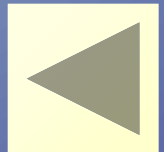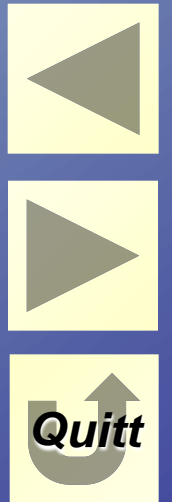
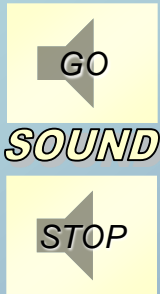
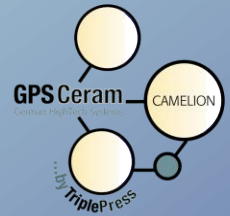
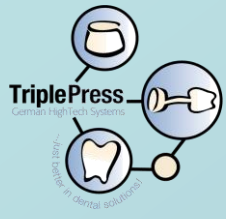
Fertige Wachsbrücke,
finished wax-up of the bridge




Im Artikulator,
finished in articulator

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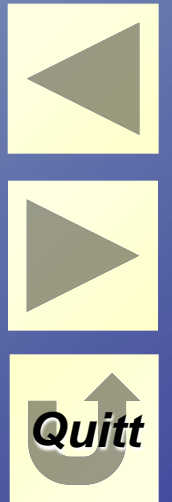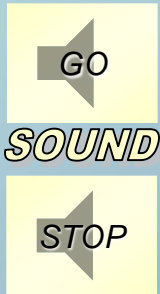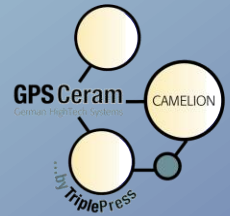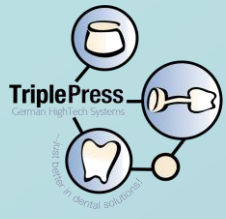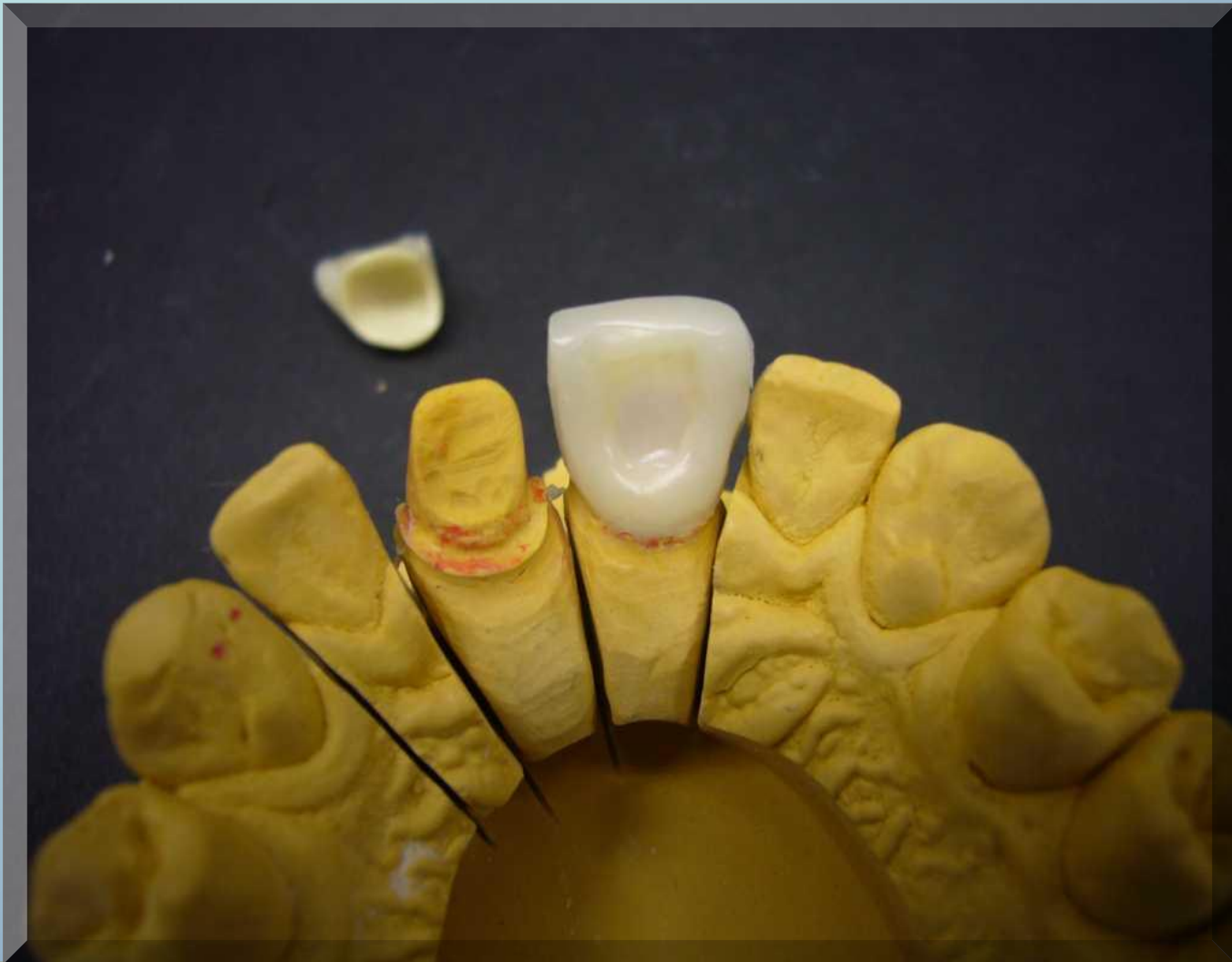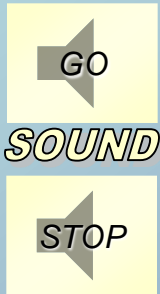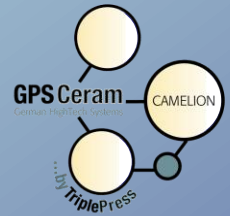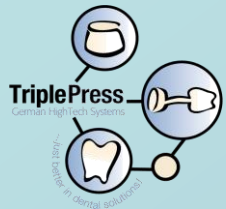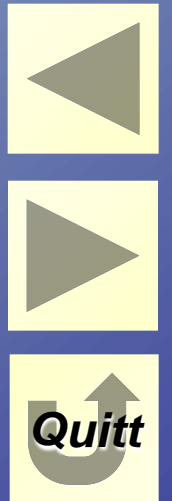
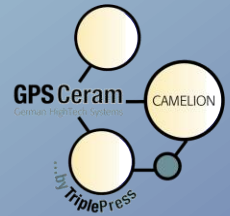
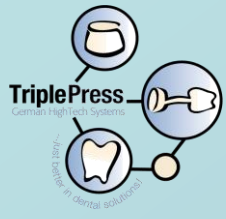
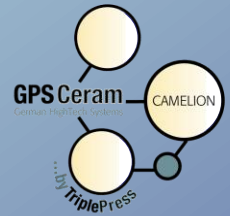
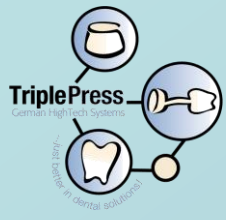
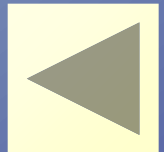
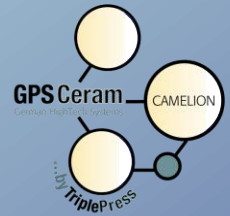
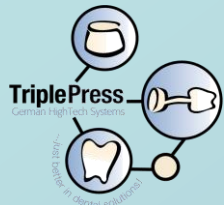
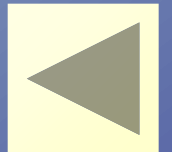
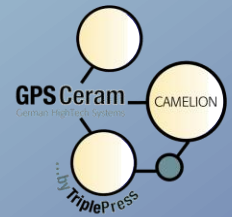
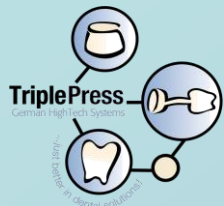
Bruttogewicht 102g minus Sockelgewicht 97g minus Brücke 3,4g


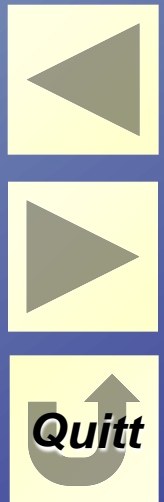
= ca. 6g Keramik


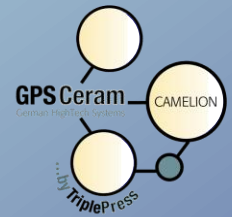
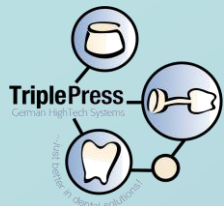
Wachskalkulation in Keramik,
calculation of needed porcelain = 6g

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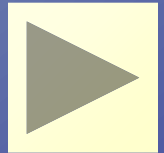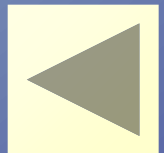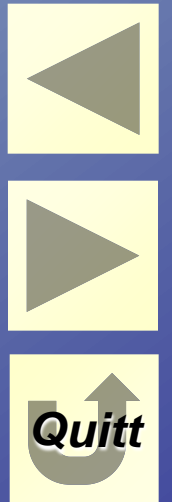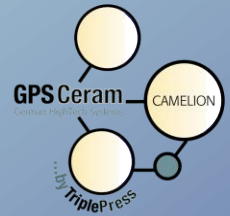
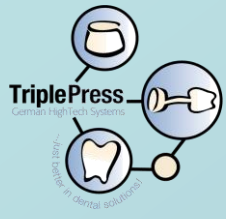
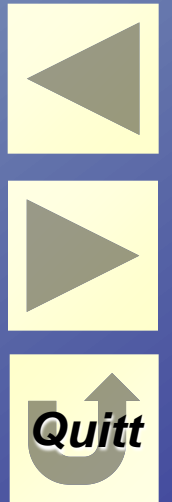
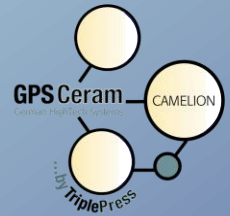
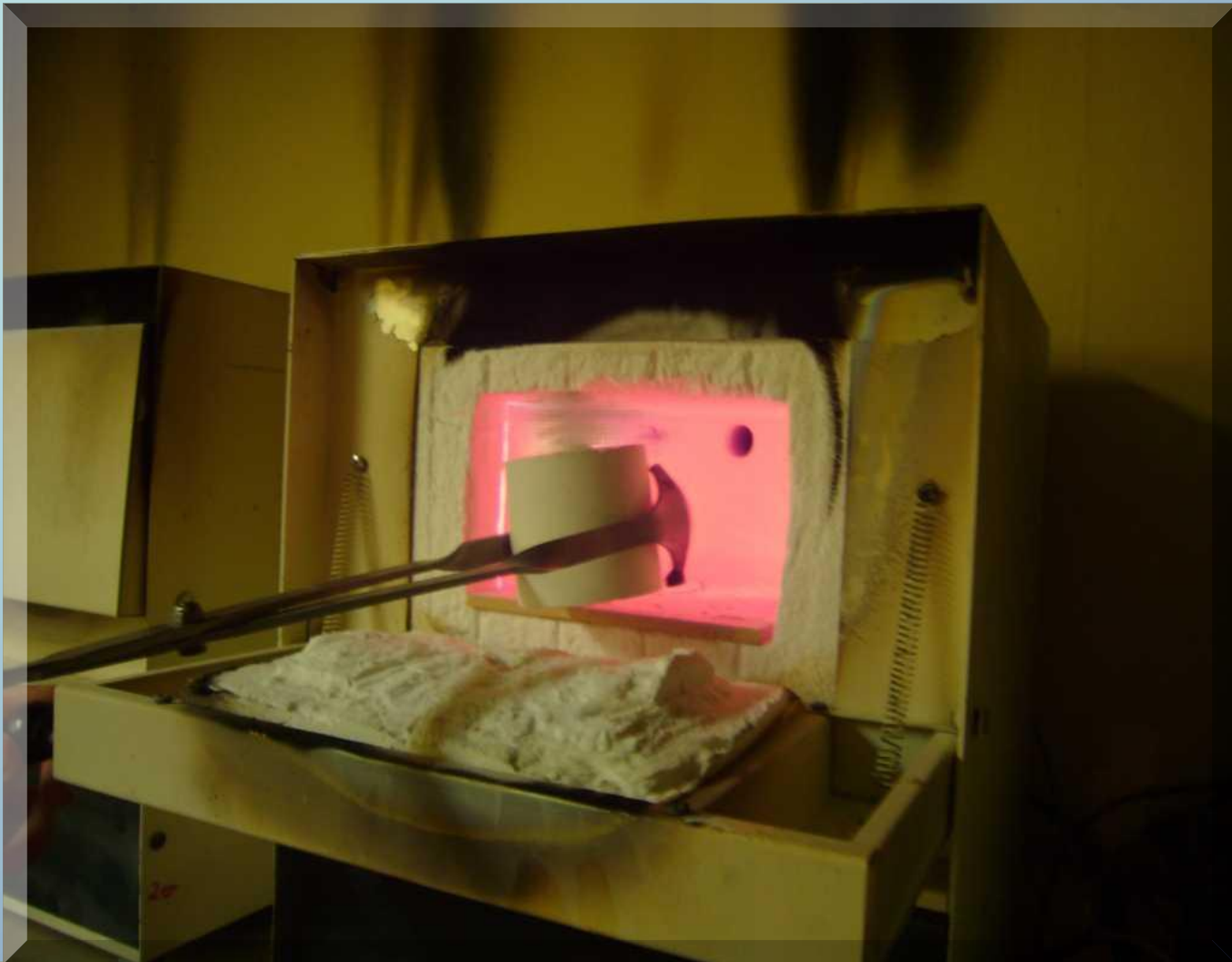
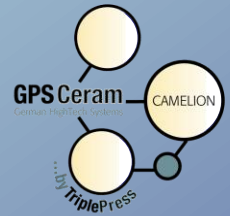
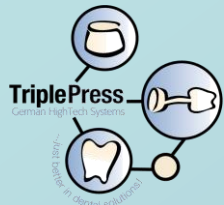
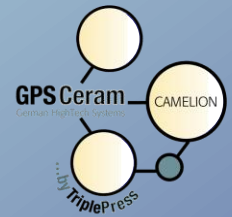
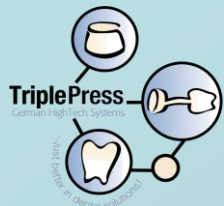
SOUND

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
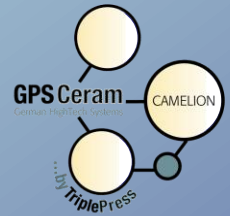
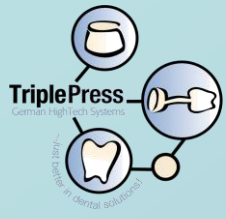
Camelion-Ingot for all shades, small 2g, large 2,5g

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

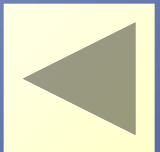
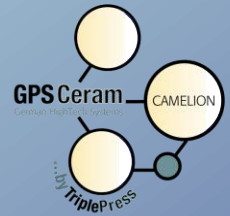
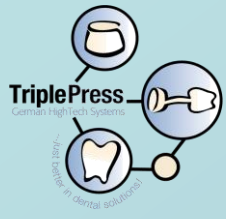
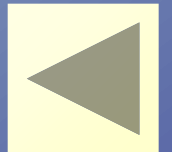
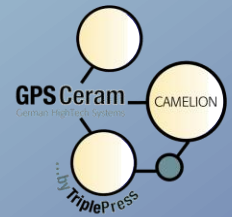
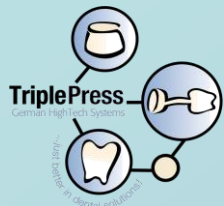
SOUND

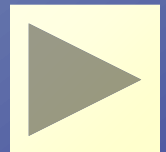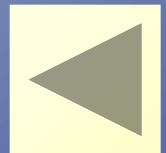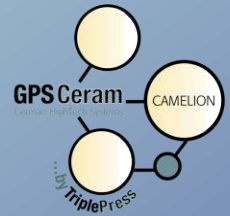
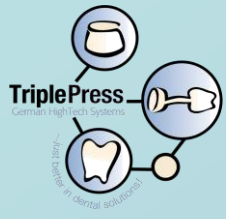
Pressung der Muffel,
pressing of the ring, use firing-charts!



SOUND


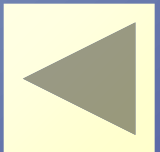
Gepresste Muffel,
pressed ring

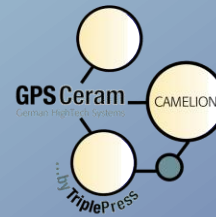
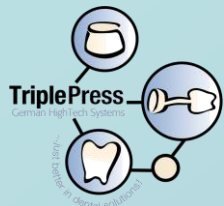
SOUND

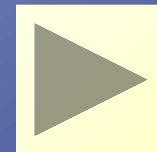
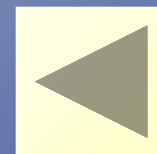
Ausbettung,
devesting

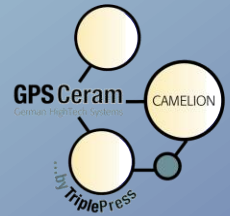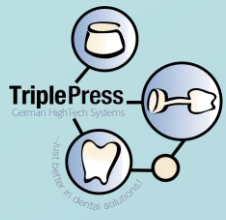
SOUND

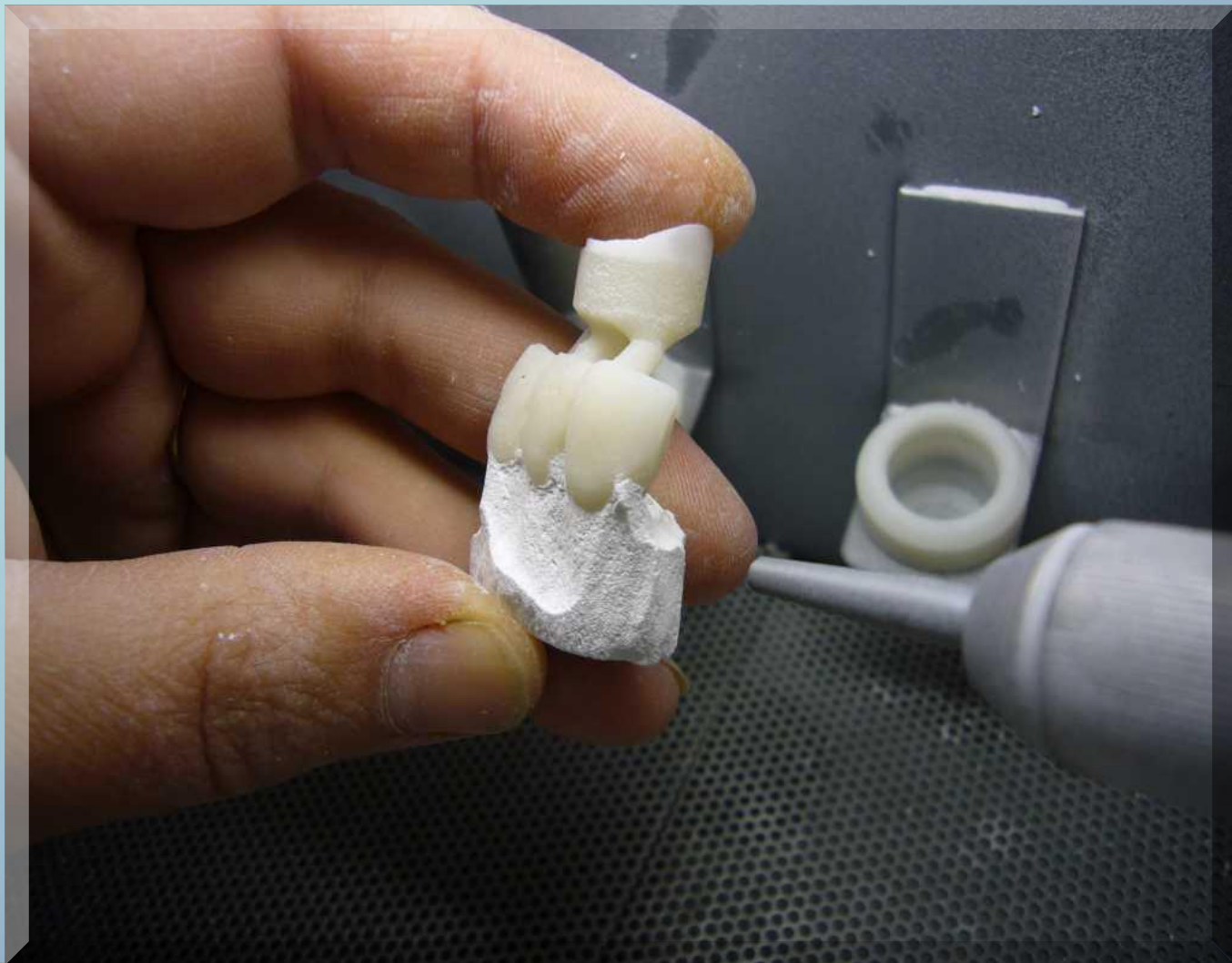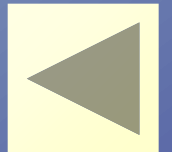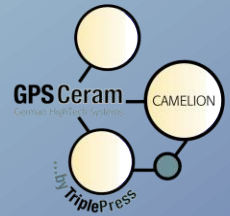
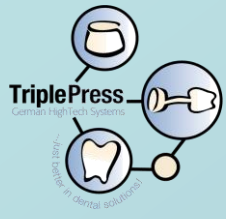
Abstrahlen 50m μ Glasperlen,
sandblasting with 50m μ glass beads


SOUND

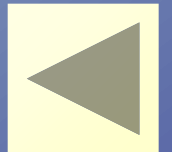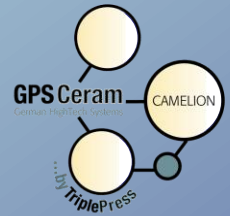
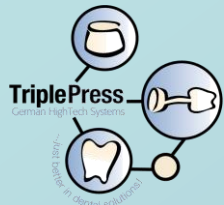
Ausbetten 50my Glasperlen,
by 50m μ glass beads


SOUND

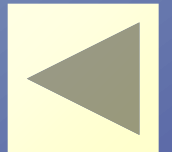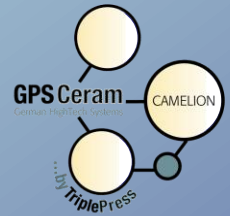
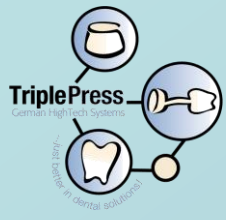
Gepresste Brücke,
pressed bridge devested

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



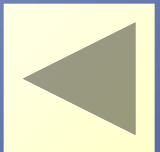
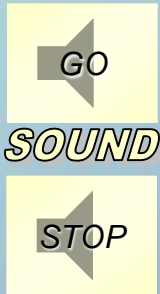
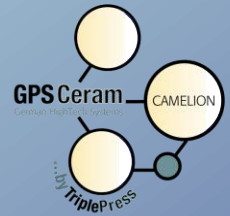
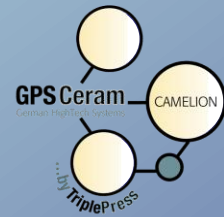
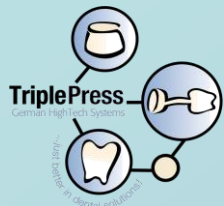
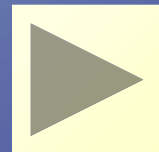
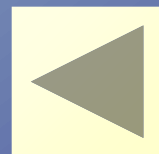
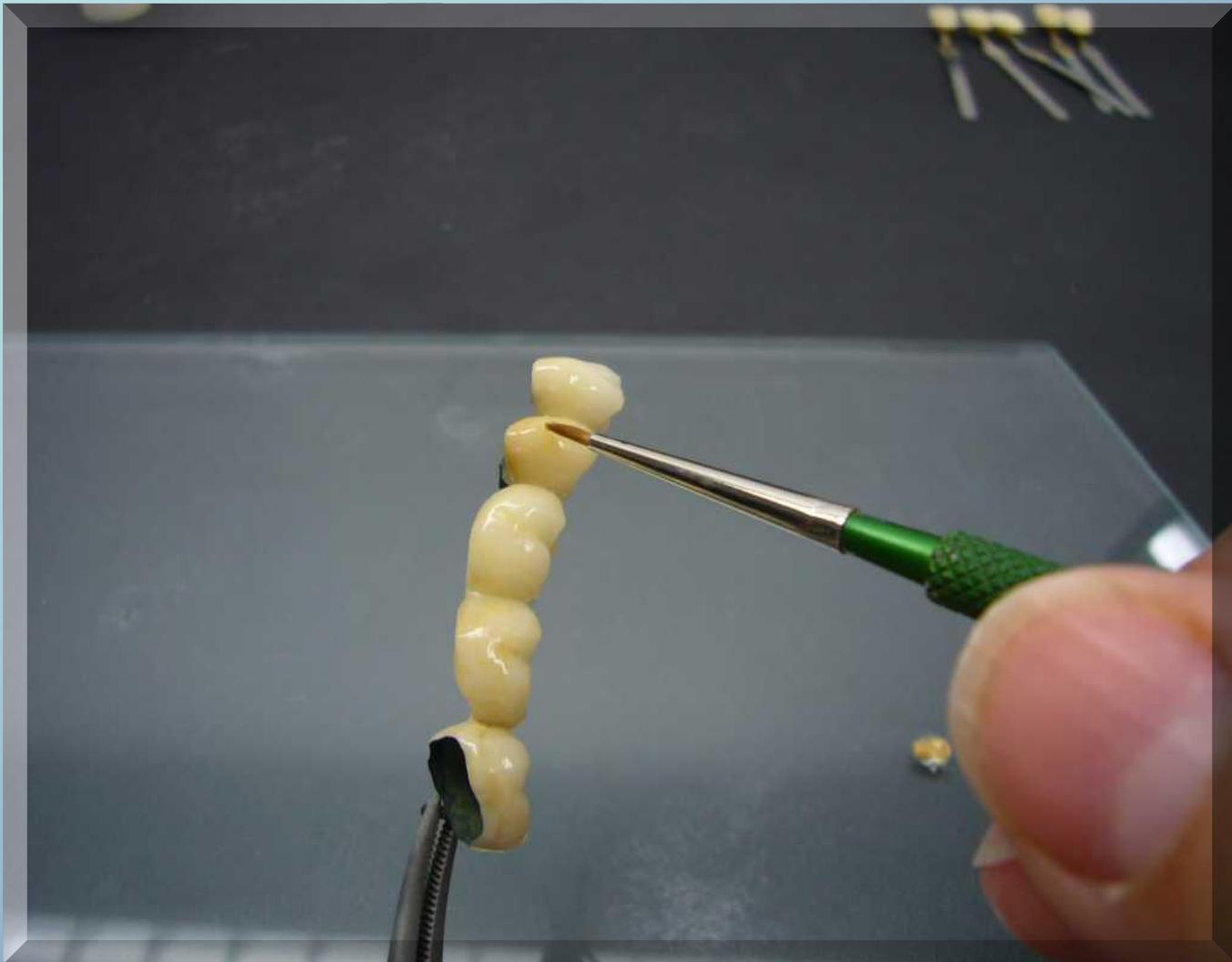
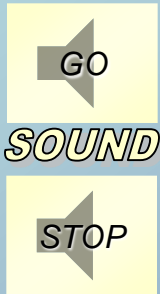
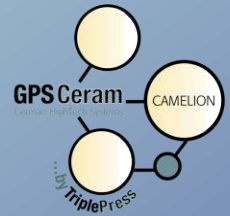
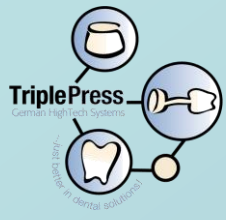
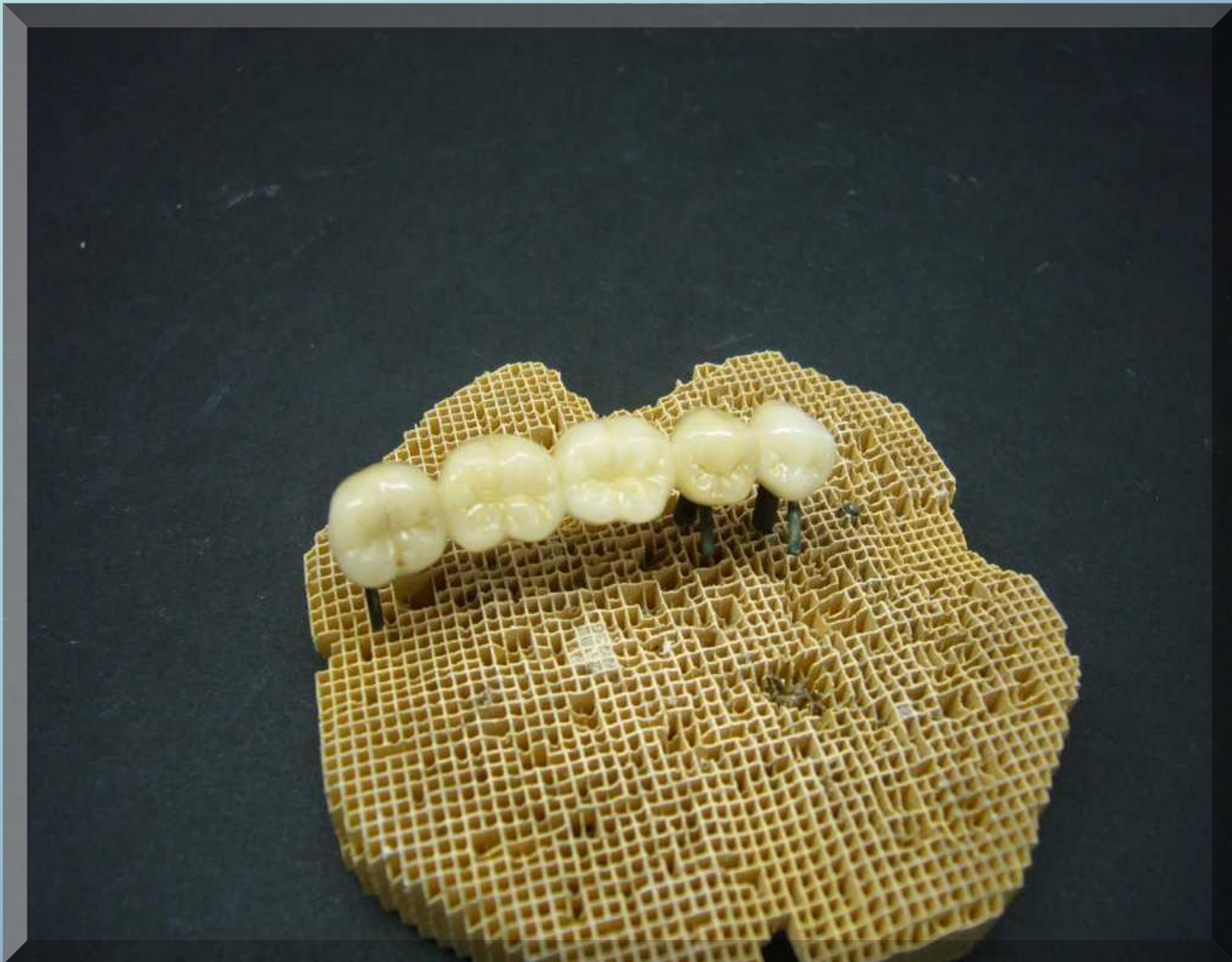
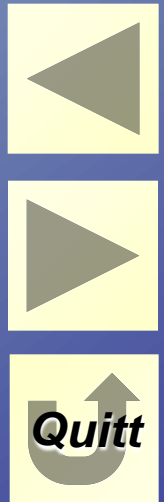
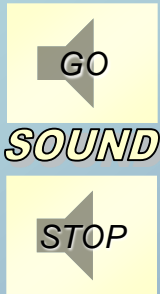
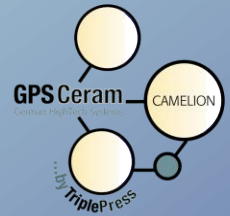
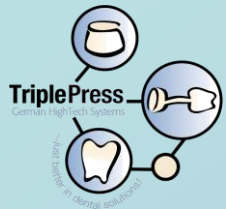
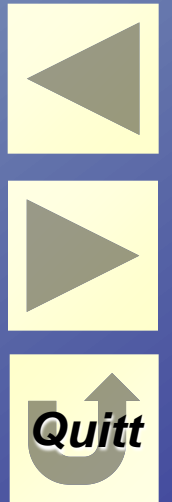
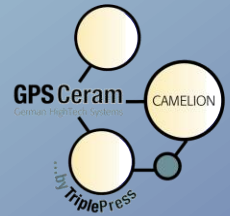
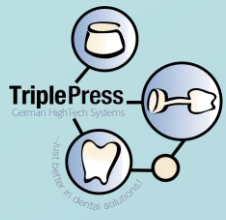
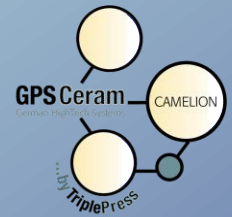
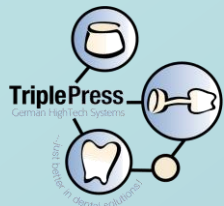
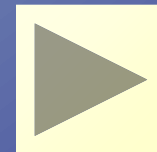
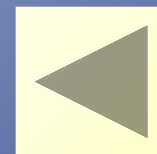
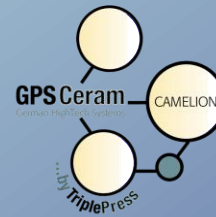
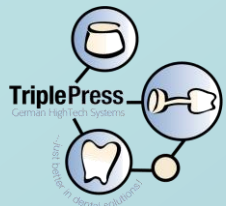
Perfekte Kronen,
perfect crowns

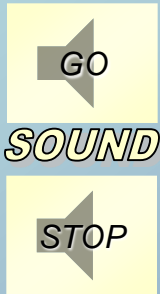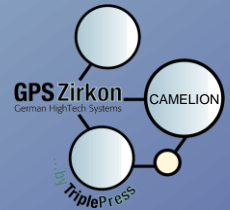
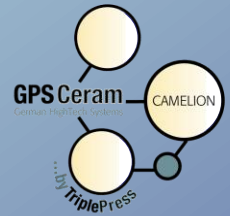
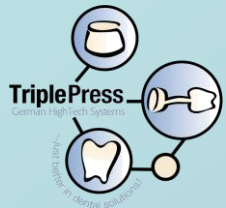
Bemalte Front,
stained only one time!

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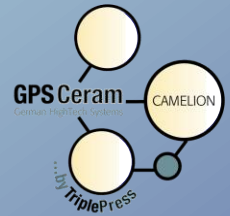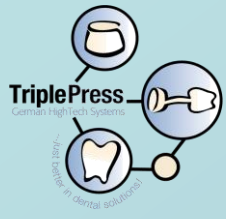
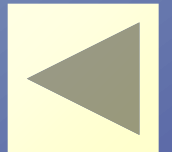
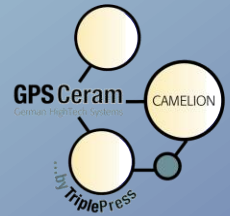
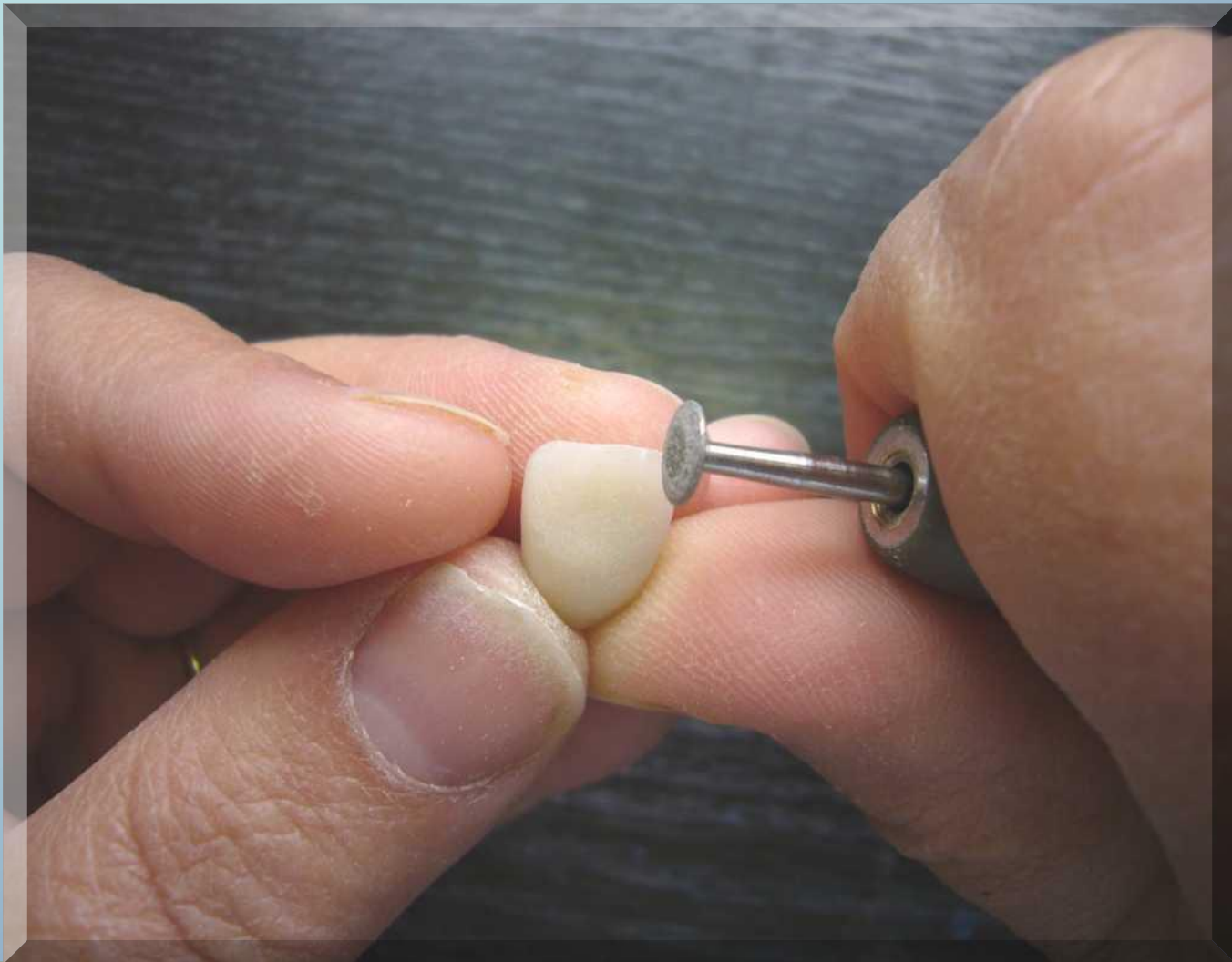
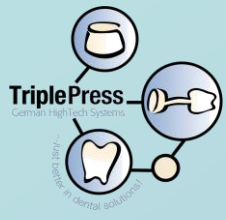
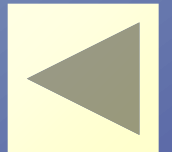
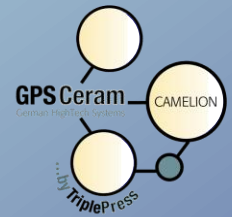
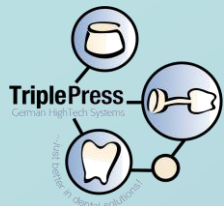
SOUND

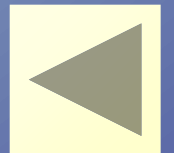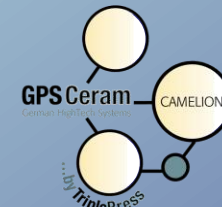
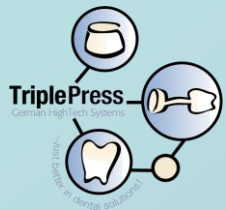
Camelion- Individualmassen 780°C S58, S59, Transpa.,
Camelion-Individual powders, S58, S59,
Transpa., 780°C use firingcharts!

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
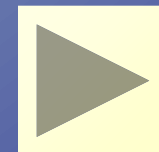
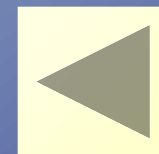
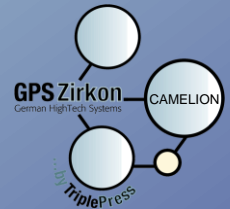
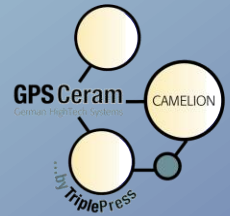
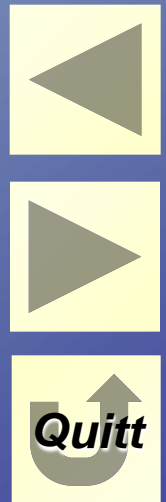
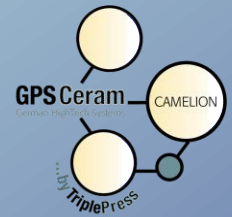
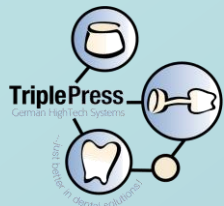
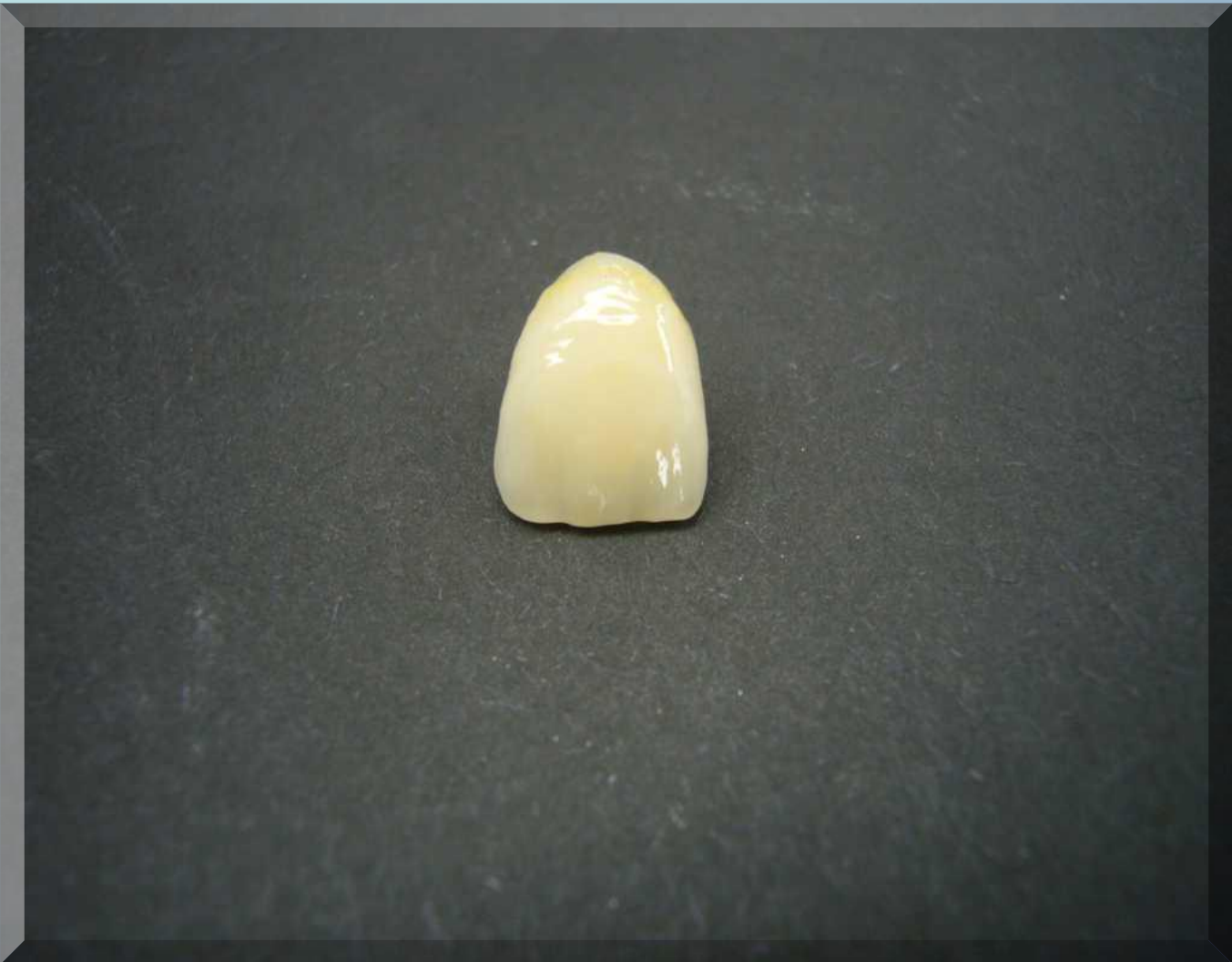
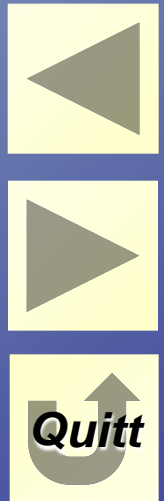
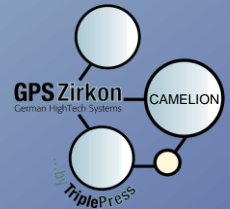
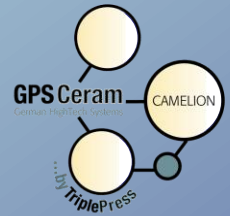
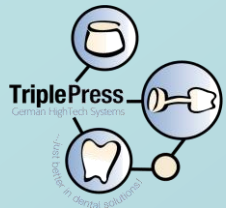
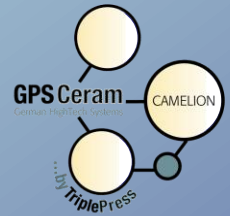
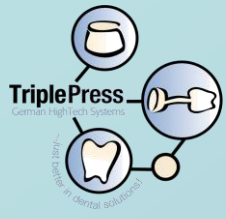
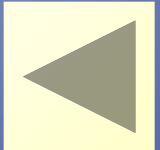
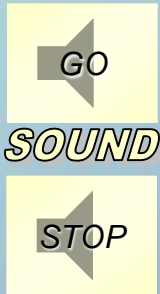
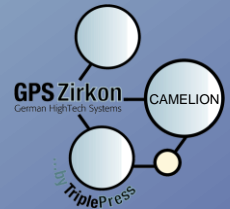
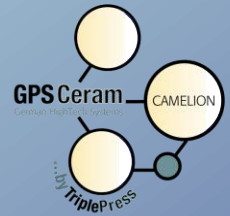
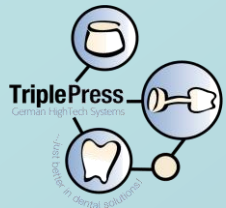
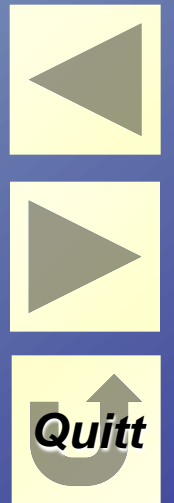
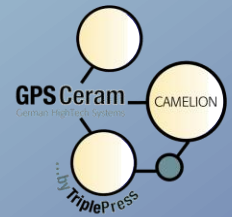
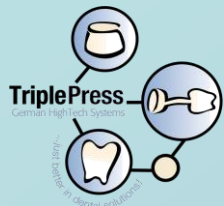
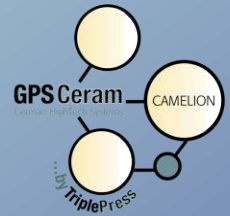
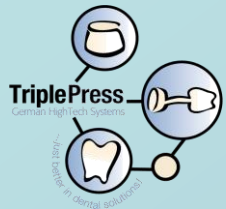
GPSceram-Inlay gemalt,
stained GPSceram-inlay






SOUND

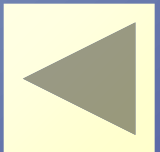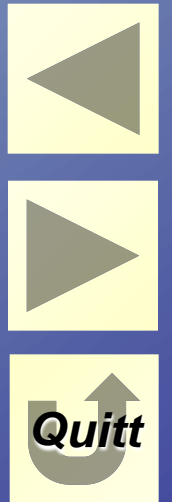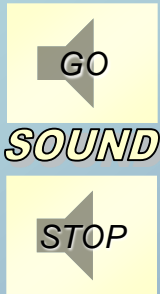
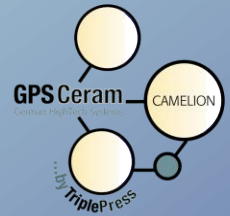
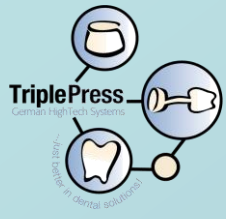
GPSceram-Veneer gemalt,
stained GPSceram-veneer

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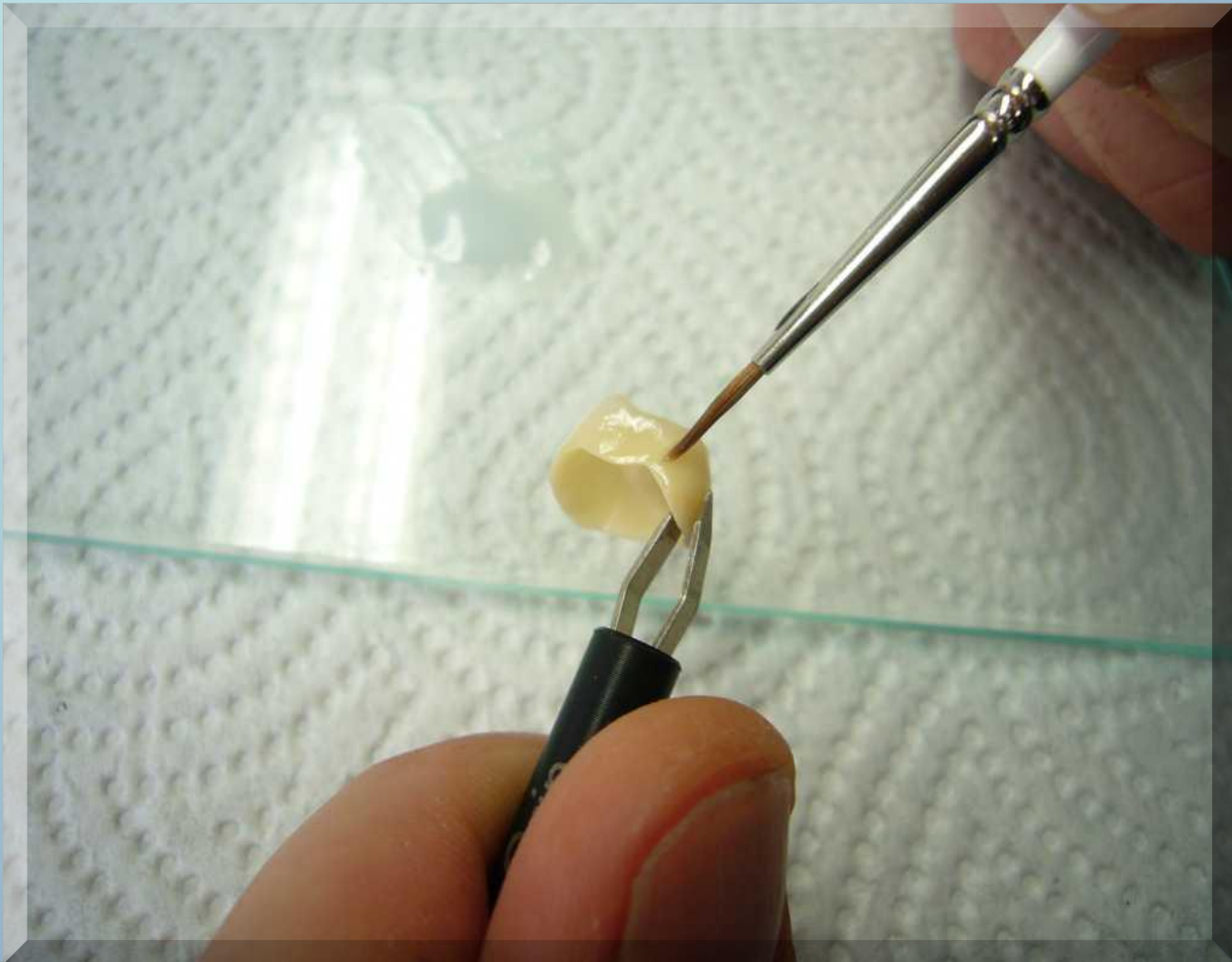
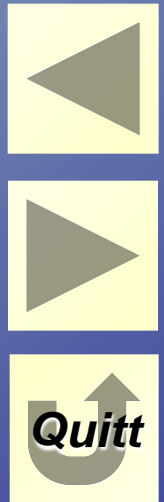
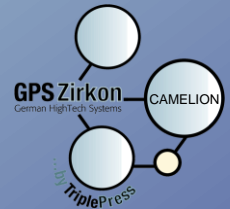
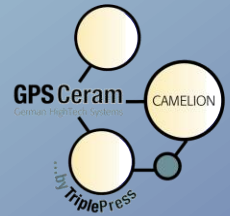
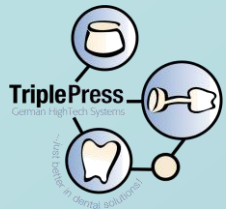
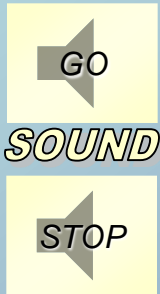
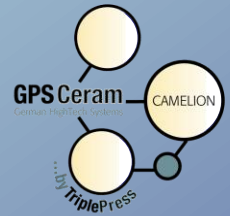
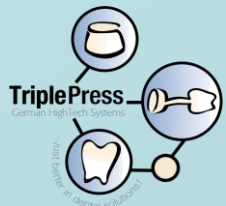
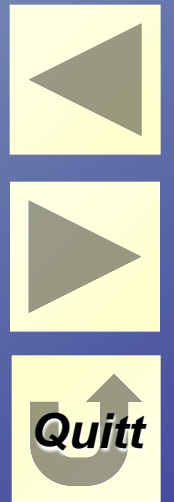
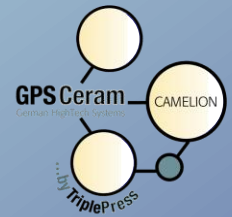
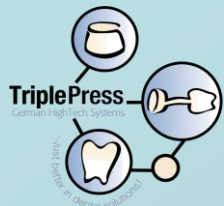
perfekte Keramikschulter,
Perfect porcelain margin

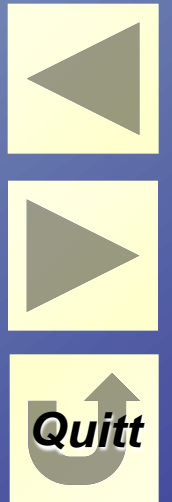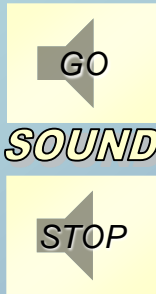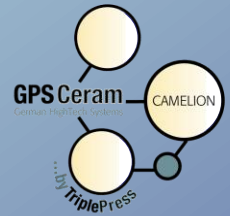
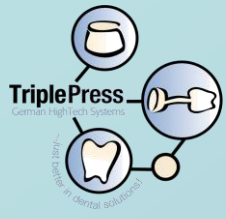
Zirkonbrücke,
bridge pressed by GPSzirkon,
staining and glaze bake at 750°C

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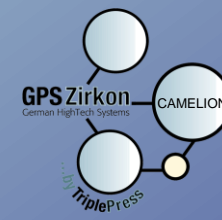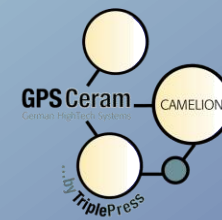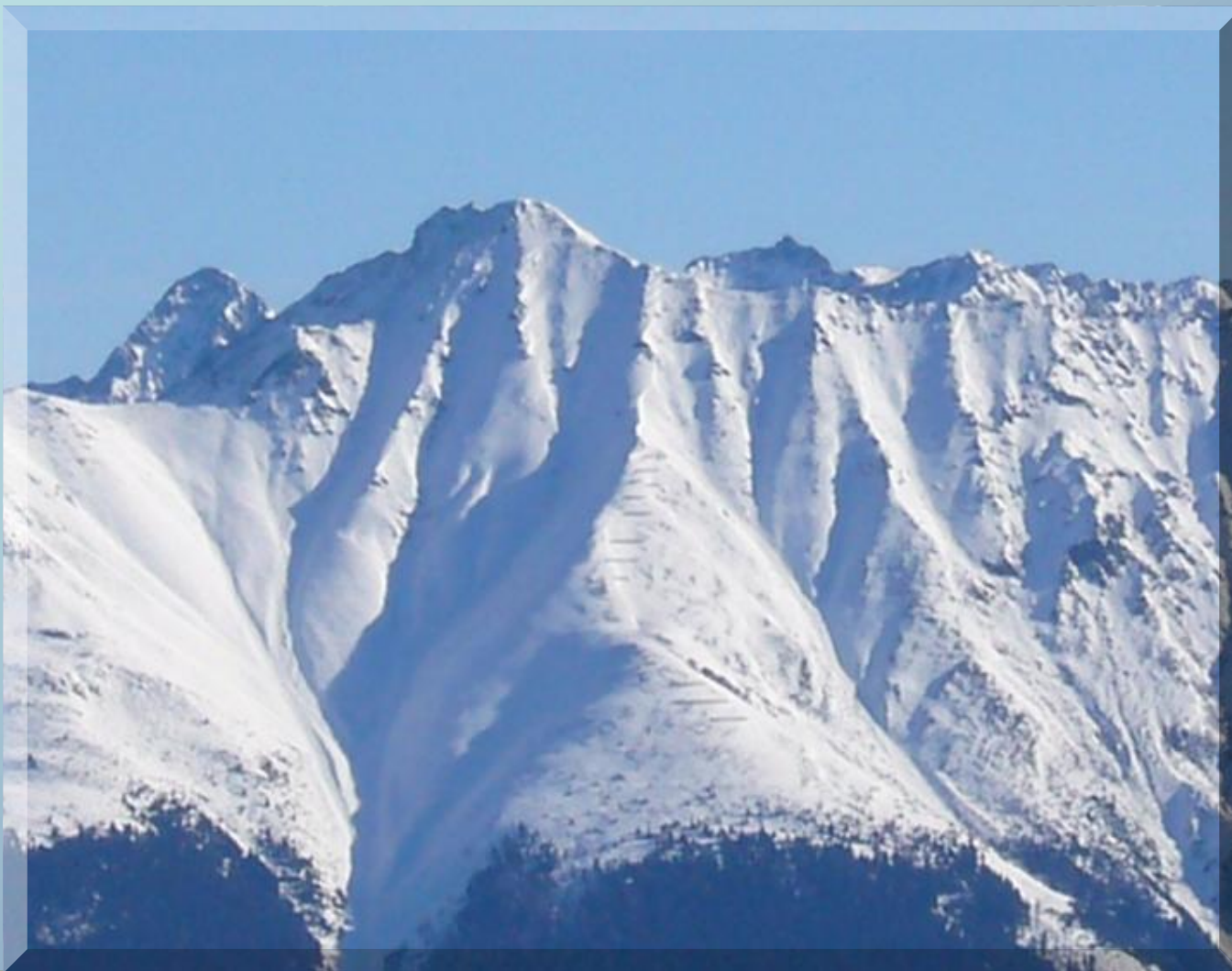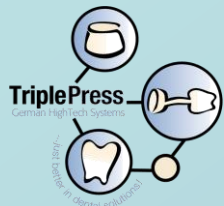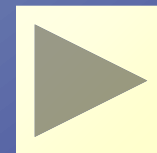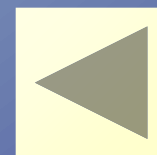
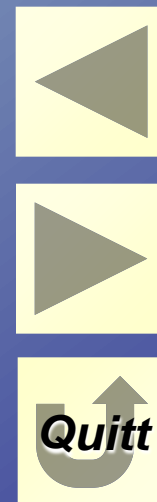
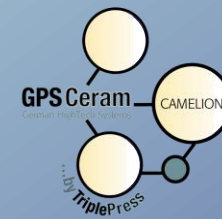
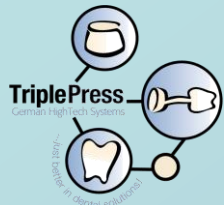
lower labor costs, higher output by up to 30-50%

TriplePress

German HighTech Systems

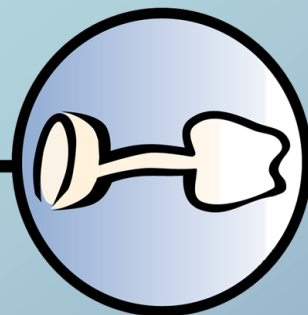...just better in dental solutions!

GPS Ceram
German HighTech Systems

CAMELION

...by TriplePress

GPS Zirkon
German HighTech Systems

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Temp. Chart
温度换算

For correction powder & individual powder 修整烧结


Firing temp 烧结温度	B 400	t^ 60	T 810° C	S 6	H 1	V1 550° C	V2 810° C
----------------------------	-----------------	-----------------	-----------------------	---------------	---------------	------------------------	------------------------

Celsius 摄氏温度	Fahrenheit 华氏温度
400	752
550	1022
700	1292
780	1436
900	1652
930	1706

Glazing paste 上釉烧结

Firing temp 烧结温度	B 400	t^ 60	T 780° C	S 4	H 2	V1 550° C	V2 780° C
----------------------------	-----------------	-----------------	-----------------------	---------------	---------------	------------------------	------------------------

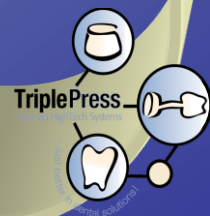*firing temperatures can vary. Please check visual result.
One may need to adjust temp.
烧结温度可根据外观效果调节

Firing Chart for: GPS Zirkon

Press cycle	B	t [^]	T	N	H	V1	V2
	700	60	850 ^C	2	22	750 ^C	850 ^C

For correction powder & individual powder


Firing temp	B	t [^]	T	S	H	V1	V2
	400	60	810 ^C	6	1	550 ^C	810 ^C

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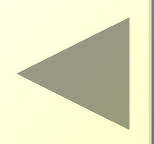SOUND


STOP

Firing Chart for GPSceram Camelion

<u><i>GiAlloy CB</i></u>	<u><i>Perfektpress AU</i></u>	<u><i>Argen Bio 86 PF</i></u>
Sandblast with 100 microns alumina- oxide	Sandblast with 100 microns alumina- oxide	Sandblast with 100 microns alumina-oxide
Degas B t^ T S H 400 55 1050 ^C 1 5	Degas B t^ T S H 550 55 1050 ^C 1 5	Degas B t^ T S H 550 55 1050 ^C 1 5
Sandblast with 100 microns alumina oxide		
Opaquer B t^ T S H V1 V2 400 55 970 ^C 8 1 550 970	Opaquer B t^ T S H V1 V2 400 55 970 ^C 8 1 550 970	Opaquer B t^ T S H V1 V2 400 55 970 ^C 8 1 570 970
Pressprogram B t^ T N H V1 V2 700 60 950 ^C 2 22 700 950	Pressprogram B t^ T N H V1 V2 700 60 950 ^C 2 22 700 950	Pressprogram B t^ T N H V1 V2 700 60 950 ^C 2 22 700 950
Glaze B t^ T S H V1 V2 400 60 900 ^C 6 1 580 900	Glaze B t^ T S H V1 V2 400 60 900 ^C 6 1 580 900	Glaze B t^ T S H V1 V2 700 60 900 ^C 6 1 580 900
Add on B t^ T S H V1 V2 400 60 910 ^C 6 1 550 909	Add on B t^ T S H V1 V2 400 60 910 ^C 6 1 550 909	Add on B t^ T S H V1 V2 400 60 910 ^C 6 1 550 909
Glaze (with glazing paste) B t^ T S H V1 V2 400 60 780 ^C 4 2 580 780	Glaze (with glazing paste) B t^ T S H V1 V2 400 60 780 ^C 4 2 580 780	Glaze (with glazing paste) B t^ T S H V1 V2 400 60 780 ^C 4 2 580 780

• firing temperatures can vary. Please check visual result. One may need to adjust temp.

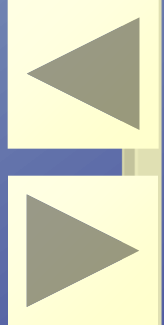* based on Ivoclar P95

TriplePress

German HighTech Systems

...just better in dental solutions!

GO

SOUND

STOP

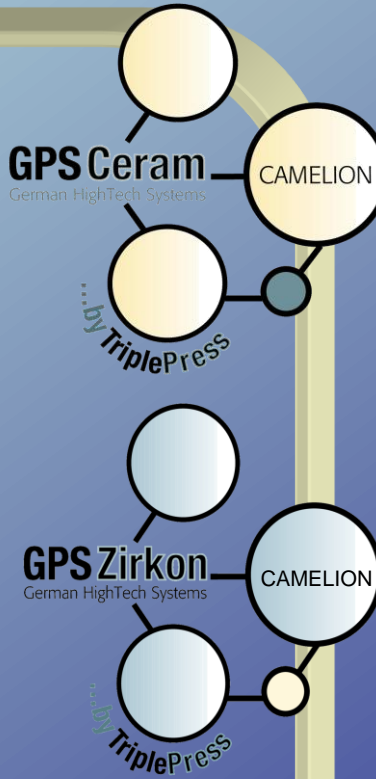
deutsch

english

©TriplePress

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Advantages of GPSCeram and GPSZirconia

- The GPS-System saves time and costs. These savings can be matched up to 30%-50%!
- Only one Camelon Ingot for 16 Shades
- Only one Camelon Ingot for Non Precious, Au, Veneers, Inlays, small Bridges
- Only Opaquer – 970°C
- Changing of shade after finish possible
- No complicated layering necessary
- Several shades in one ring, for example A,B,C,D (VITA)
- Correction Powder Camelon 905°C – 910°C
- Big bridges possible
- Individual layering possible, but not necessary
- Individual LFC Powders 780C (also for corrections in margin area)
- LFC-Staining shades A,B,C,D,K 780°C for all ceramics
- Perfect porcelain margins
- K-Shade for staining the ceramic shoulder from inside and outside possible
- Use Glazing paste for Veneers, Inlays and correction of margin
- Bleach Ingot for shades brighter than B1
- High stability while grinding
- At 780°C glazing and staining bake
- Only wax-up → canceling possible all the time
- Apprentice can wax-up, correction every time possible

GO

SOUND

STOP

- Try-In by dentist → occlusion in wax (Gnatology)
- Reproduction Wax 1:1 Ceramic
- No shrinking of the ceramic after being pressed
- Speed-Investment for Non Precious, AU, Ceramic
- Special Non Precious → Gialloy CB
- Small assortment → lower costs

Zirconia

- Only one Zirconia Ingot for all 16 shades (VITA)
- No Liner necessary
- Layering possible, but not necessary
- Optimizing the shoulders
- Individual Correction powders on LFC basis → Enamel and effect powders
- LFC Staining shades A,B,C,D,K
- Correction with Zirconia- Camelion shades possible
- No shrinking of ceramic after pressing
- Bleach Ingot for all shades brighter than B1
- Small assortment → lower costs

SOUND

Wax Forms:

- Fast, constant quality, different forms,
- pontic waxed only one time → finished
- Wax-Up with Hairdryer
- Wax try-in 37C in patients mouth
- Reproduction Wax 1:1 Ceramic

Special

- Wax-Up with hair dryer
 - Metal pin for holding the framework, pin inside the framework usefull, stability of the ring
 - 0,6gr Wax = 2gr Ceramic
 - Pay attention to a precise investing (No Bubbles!)
- The GPS-System saves time and costs. These savings can be matched up to 30% - 50%!

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